

### Intended Use

The Sterilization Kit eliminates the need to secure the neural implant inside the coupler during surgery.


### Materials Needed

- + Neural implant
- + Coupler
- + Materials needed to secure the neural implant to the coupler
- + Sterilization Kit

### Setup Instructions

1. Secure the neural implant to the coupler.  
**Note:** These instructions depict an Array Clamp Coupler. For instructions on how to use different coupler types, visit [neuralglider.com/support](http://neuralglider.com/support).
2. Place the Array Clamp Luer onto the Sterilization Kit Luer and twist the Sterilization Kit Luer until tight.
3. Close the box and place it in a sterilization pouch. Sterilize according to institution protocol.

1

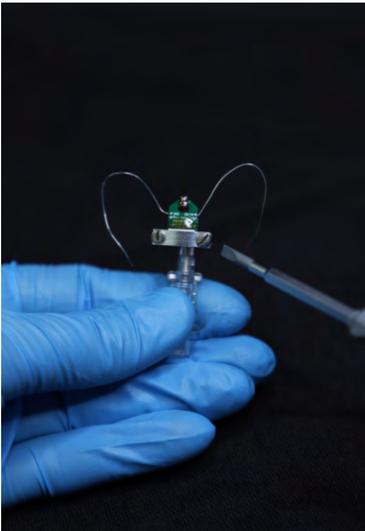

2



3



### Operator Assistance Information

If you have questions regarding our product, please contact:

Actuated Medical, Inc. Customer Service Department

Phone +1 (814) 355-0003 ext. 117

Fax +1 (814) 355-1532

Monday through Friday

8:00 am - 5:00 pm U.S.A. Eastern Standard Time



Innovative motion + Positive outcomes



Learn more at  
[NeuralGlider.com](http://NeuralGlider.com)

These works were supported by the Brain Initiative at the NIH/NINDS SBIR Grant No. NS105500, NIH/NIDA SBIR Grant No. DA051265 and DARPA BTO Electrical Prescriptions Contract No. HR0011-16-C-0094. The content is solely the responsibility of the authors and does not represent the official views of the government.



intertek



certified  
SBA WOSB  
Woman Owned Small Business



certified  
WBENC  
WOMEN BUSINESS ENTERPRISE



AdviaMed Code of Ethics  
CERTIFIED

1100794338-000

Pat. [actuatedmedical.com/ip](http://actuatedmedical.com/ip)